Hyalobarrier [®] <small>GEL</small>	Hyalobarrier [®] <small>GEL</small>	Hyalobarrier [®] <small>GEL</small>	Hyalobarrier [®] <small>GEL</small>
STERILE <small>H₂O</small>	Sterilized using vaporized hydrogen peroxide	STERILE <small>H₂O</small>	Sterilisation au peroxyde d'hydrogène vaporisé
STERILE <small>A</small>	Sterilized using steam	STERILE <small>A</small>	Sterilisation à la vapeur d'eau
STERILE <small>E</small>	Sterilized using ethylene oxide	STERILE <small>E</small>	Sterilisation à l'éthylène oxydane

STERILE <small>H₂O</small>	Sterilized using vaporized hydrogen peroxide
STERILE <small>A</small>	Sterilized using steam
STERILE <small>E</small>	Sterilized using ethylene oxide

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Product information

Abdomino-pelvic surgical procedures are a frequent cause of adhesion formation, which may induce pelvic pain and/or infertility (1). These post-surgical adhesions are due to the formation of areas of contact between adjacent internal organs. In order to prevent the formation of post-surgical adhesions it is recommended to use a product able to form a barrier against the contact between adjacent tissues, and to remain at the site of application for a period of approximately 6 to 8 weeks (2).

The medical devices HYALOBARRIER[®] GEL and HYALOBARRIER[®] GEL ENDO are sterile, transparent and highly viscous gels, based on ACP[®] (auto-crosslinked polysaccharide), which is obtained through crosslinking by condensation of hyaluronic acid, one of the main components of human connective tissue and of epithelial and mesothelial tissues. Thanks to their viscosity, HYALOBARRIER[®] GEL and HYALOBARRIER[®] GEL ENDO perfectly adhere to the tissue to be protected, creating an anti-adhesion barrier which keeps the adjacent tissues separated during the repair phase subsequent to a surgical procedure. Seven days after application, the gel is completely reabsorbed (3). The efficacy of the products has been demonstrated in clinical studies (4-13) performed in abdomino-pelvic surgeries. HYALOBARRIER[®] GEL and HYALOBARRIER[®] GEL ENDO are designed to satisfy the needs of the different techniques used in abdomino-pelvic surgery.

Indications

HYALOBARRIER[®] GEL and HYALOBARRIER[®] GEL ENDO are indicated for the prevention or reduction of post-surgical adhesion formation in the abdomino-pelvic area. HYALOBARRIER[®] GEL is indicated for use in open surgery procedures. HYALOBARRIER[®] GEL ENDO is indicated for use in laparoscopic and hysteroscopic surgical procedures.

Contraindications

The following pre-existing conditions may constitute absolute or relative contraindications to the use of the product:

- Known hypersensitivity to the component of the product, hyaluronan or its derivatives.
- Patients who should not be used in patients with infection or contamination of the surgical site.
- Any individual including pregnant woman, nursing mother or lactating women, should not be used in patients with infection or contamination of the surgical site.
- Patients affected with malignancy or alterations of blood coagulation have not been evaluated with the use of the products.

- Keep refrigerated (2-8°C). The product may be stored at room temperature only for a limited time (no longer than 7 days), after which it should be refrigerated again.
- Do not freeze.

Instructions for use

HYALOBARRIER[®] GEL and HYALOBARRIER[®] GEL ENDO are provided in sterile single-use, pre-filled syringes. The syringe content is sterilized using steam. The pouch with syringe is sterilized using vaporized hydrogen peroxide to assure sterility of the whole pouch content, including outer syringe surface. Cannulas are sterilized by ethylene oxide.

1. Open the product out of refrigeration and let it warm to room temperature (minimum 30-40 minutes in ambient room temperature). Open the pouch and introduce the syringe into the sterile operating field, adopting the normal aseptic techniques used in minimally-invasive surgical procedures.
2. Open the cannula's blister and transfer the cannula into the sterile field, adopting the normal aseptic techniques used in the surgical field.
3. Attach the syringe protective cap on the tip of the syringe. With a twisting motion, connect the tip of the Luer-lock end of the syringe. Gently tug on the cannula. If the cannula is properly fitted on the syringe, it should not pull off. The cannula for HYALOBARRIER[®] GEL ENDO has been designed to be utilized with a 0.5 cm diameter trocar.
4. Apply the gel inside the abdomino-pelvic cavity by pushing the plunger.
5. Once the area is treated with the gel, it is recommended to apply a 1-2 mm thick layer of gel. The thickness of the gel layer does not influence the efficacy of the product.
6. In case of pack with 2 syringes and 1 cannula, if a second syringe is used, it should be used to fill the syringe of the cannula in the end of the follow 3 steps above to attach the second syringe to the cannula.
7. Do not irrigate the surgical field after application of the product.

1. Wash the hands to be treated with the gel. It is recommended to apply a 1-2 mm thick layer of gel. The thickness of the gel layer does not influence the efficacy of the product.
2. Apply the gel inside the abdomino-pelvic cavity by pushing the plunger.
3. Once the area is treated with the gel, it is recommended to apply a 1-2 mm thick layer of gel. The thickness of the gel layer does not influence the efficacy of the product.
4. In case of pack with 2 syringes and 1 cannula, if a second syringe is used, it should be used to fill the syringe of the cannula in the end of the follow 3 steps above to attach the second syringe to the cannula.
7. Do not irrigate the surgical field after application of the product.

Warnings and precautions

HYALOBARRIER[®] GEL and HYALOBARRIER[®] GEL ENDO are intended for specialized surgeons only.

- HYALOBARRIER[®] GEL and HYALOBARRIER[®] GEL ENDO must not be injected intravenously.
- The products have to be managed with sterile instruments.
- HYALOBARRIER[®] GEL and HYALOBARRIER[®] GEL ENDO do not have intrinsic bacteriostatic or bactericidal activity.
- The concomitant use of HYALOBARRIER[®] GEL ENDO with other antiseptic devices or with intraperitoneally instilled solutions has not been evaluated.
- It is recommended to avoid pregnancy during the first complete menstrual cycle subsequent to the treatment.
- It is recommended to use the syringe and cannula immediately after opening the pouch.
- The aseptic operations of the device must be performed in the sterile operating field.
- To avoid damage to the Luer-look connection, it is recommended not to use the cannula-syringe system as a manipulating laparoscopic instrument.
- The syringe is single-use; any non-used product must be discarded.
- The cannula is single-use; do not sterilize again.
- Do not use the product after the expiration date.
- The empty containers have to be discarded according to national regulations.
- If the protective pouch or the blister is damaged, do not use the product and contact local distributor.
- Do not use the product after the expiration date.

How supplied

In each pack, HYALOBARRIER[®] GEL is available in individually packaged syringes each containing 10 ml of 40 mg ACP/ml sterile gel. Individually packaged 5-cm long cannulas are enclosed for laparoscopic use.

- In each pack, HYALOBARRIER[®] GEL ENDO is available in individually packaged syringes each containing 10 ml of 30 mg ACP/ml sterile gel. Individually packaged 30-cm long cannulas are enclosed for application.
- Each pack contains 2 syringes and 1 cannula. If a second syringe is used, it should be used to fill the syringe of the cannula in the end of the follow 3 steps above to attach the second syringe to the cannula.
- Do not irrigate the surgical field after application of the product.

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